



Innovation in Mobility

***A6000 Series
Personal Use
Wheelchair Lift***

Service/Owner Manual

(Retained by Owner After Installation by an Authorized RICON Service Technician)

10/02/00

32DA6P00.G

©1999-2000 RICON CORP.

All Rights Reserved

Printed in the

United States of America

U.S. Patent # 5,308,215;
Other U.S. and Foreign Patent(s) Pending.

This RICON product must be installed and serviced only by authorized RICON service technicians.

The authorized RICON service technicians must refer to this manual for installation instructions.

The owner must refer to this manual for operating instructions, then retain it for future reference by authorized RICON service technicians who perform service and repairs.

Customer Name:	_____
Installing Dealer:	_____
Date Installed:	_____
Serial Number:	_____

REVISION RECORD

REV #	PAGE(S)	DESCRIPTION OF CHANGE	ECR / ECO #
32DA6P0 0.G 10/02/00	4-2	Changed hydraulic oil to Texaco 01554 per mil H5606G.	3215/4583
	4-11	Changed CB amp values in Figure 4-5, sheet 2; pressure switch relay redrawn.	3280/4583
	3-3	Added paragraph D.2 "Handrail up/down switch".	None/4583
	1-4, 1-5	Created a Lift Components figure and Table	3354/4583
	2-8	Figure 2-8 (Platform tie rod linkage) was incorrect. Replaced with correct illustration.	
	Chapter 5	Renumbered figures in all spare parts drawings.	
	5-6, 5-7	Added kit #19519 (Kit, Platform Tie Rod Assy.) To figure 5-3	
END OF TABLE			

I. INTRODUCTION	1-1
RICON 5-YEAR LIMITED WARRANTY	1-2
SHIPMENT INFORMATION	1-3
GENERAL SAFETY PRECAUTIONS	1-3
II. INSTALLATION	2-1
A. MECHANICAL INSTALLATION	2-1
1. LIFT LOCATION	2-1
2. LIFT INSTALLATION GUIDELINES	2-2
3. LIFT INSTALLATION INTO VANS	2-2
B. ELECTRICAL INSTALLATION	2-3
1. MAIN CIRCUIT BREAKER INSTALLATION	2-4
2. CONTROL INTERFACE AND DOOR OPERATOR CONNECTIONS	2-4
3. MAIN POWER CABLE ROUTING AND CONNECTION	2-5
4. LIFT GROUNDING INSTRUCTIONS	2-6
C. FINAL ADJUSTMENTS	2-6
1. LIMIT SWITCH ADJUSTMENT	2-6
2. PLATFORM TILT (ROLLSTOP) ADJUSTMENT	2-7
3. LINKAGE ADJUSTMENT FOR FOLDING PLATFORM	2-8
D. VERIFY INSTALLATION	2-9
E. CUSTOMER ORIENTATION	2-9
III. OPERATING INSTRUCTIONS	3-1
A. SAFETY PRECAUTIONS	3-1
B. DAILY SAFETY CHECK	3-2
C. LIFT FUNCTIONS	3-3
D. CONTROLS AND INDICATORS	3-3
1. CONTROL PENDANT	3-3
2. HANDRAIL SWITCH	3-3
3. CIRCUIT BREAKERS	3-4
4. MANUAL BACKUP PUMP	3-4
E. LIFT OPERATION	3-6
1. PRELIMINARY	3-6

2. ENTERING VEHICLE	3-7
3. EXITING VEHICLE	3-7
4. STOWING PLATFORM	3-8
5. MANUAL OPERATION	3-8
F. MAINTENANCE AND REPAIR	3-10
IV. MAINTENANCE AND REPAIR	4-1
A. CLEANING	4-1
B. MAINTENANCE SCHEDULE	4-1
C. TROUBLESHOOTING	4-3
D. HYDRAULIC CIRCUIT DIAGRAM	4-7
E. LIFT ELECTRICAL WIRING	4-8
1. DIAGRAM LEGEND	4-8
2. WIRING DIAGRAM	4-10
V. SPARE PARTS	5-1
APPENDIX 1. LIFT SPECIFICATIONS	5-14

LIST OF FIGURES**PAGE**

FIGURE [2-1]: VAN CLAMPING BARS	2-1
FIGURE [2-2]: BASEPLATE MOUNTING HOLES	2-2
FIGURE [2-3]: ELECTRICAL INSTALLATION DIAGRAM	2-3
FIGURE [2-4]: CONTROL INTERFACE AND POWER DOOR OPERATOR CONNECTIONS	2-4
FIGURE [2-5]: POWER CABLE ACCESS HOLES	2-5
FIGURE [2-6]: LIMIT SWITCHES DIAGRAM	2-6
FIGURE [2-7]: PLATFORM TILT ADJUSTMENT	2-8
FIGURE [2-8]: PLATFORM TIE ROD LINKAGE	2-8
FIGURE [2-9]: DECAL LOCATIONS	2-10
FIGURE [3-1]: DO NOT DEPLOY LIFT ON SLOPED GROUND	3-1
FIGURE [3-2]: MAXIMUM LOAD	3-1
FIGURE [3-3]: STAND CLEAR PRECAUTION	3-2
FIGURE [3-4]: CONTROL PENDANT	3-3
FIGURE [3-5]: HANDRAIL SWITCH	3-3
FIGURE [3-6]: MAIN CIRCUIT BREAKER	3-4
FIGURE [3-7]: MANUAL BACKUP PUMP & HANDLE	3-4
FIGURE [3-8]: MANUAL BACKUP PUMP - RAISING PLATFORM	3-5
FIGURE [3-9]: MANUAL BACKUP PUMP - LOWERING PLATFORM	3-5
FIGURE [3-10]: STOW LOCK ASSEMBLY	3-9
FIGURE [4-1]: HYDRAULIC CIRCUIT DIAGRAM	4-7
FIGURE [4-2]: ELECTRICAL SYMBOLS	4-8
FIGURE [4-3]: WIRING DIAGRAM LABELS	4-9
FIGURE [4-4]: MOLEX® CONNECTORS	4-9
FIGURE [4-5]: ELECTRICAL WIRING DIAGRAM - SHEET 1 OF 2	4-10
FIGURE [4-6]: ELECTRICAL WIRING DIAGRAM - SHEET 2 OF 2	4-11
FIGURE [5-1]: PLATFORM ASSEMBLY	5-2
FIGURE [5-2]: PLATFORM ASSEMBLY, RIGHT HAND SIDE	5-4
FIGURE [5-3]: PLATFORM ASSEMBLY, LEFT HAND SIDE	5-6
FIGURE [5-4]: TRAVELING FRAME ASSEMBLY	5-8

FIGURE [5-5]: TOWER BASE ASSEMBLY	5-10
FIGURE [5-6]: TOWER BASE ASSEMBLY, BACK VIEW	5-12

LIST OF TABLES

PAGE

TABLE [2-1]: LIMIT SWITCH ADJUSTMENT CHART	2-7
TABLE [3-1]: LIFT FUNCTIONS	3-3
TABLE [4-1]: MAINTENANCE SCHEDULE	4-1
TABLE [4-2]: TROUBLESHOOTING GUIDE	4-3
TABLE [4-3]: WIRE COLOR CODES	4-6